TheraLift

POLTRONA / SILLÓN / CHAIR

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ES Manual de instrucciones

EN Instruction manual

TM3000 · TM3001 · TM3002 · TM3003 · TM3004

TheraMart

TheraLift

POLTRONA / SILLÓN / CHAIR

PORTUGUÊS

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
Assembly instructions


Please read the instructions carefully before assembly. Check all boxes thoroughly for small parts that may have come loose during transportation. Remove all components from the boxes and place them on a soft surface such as a carpet, blanket, or furniture pad. Identify and verify all parts and hardware as specified below.

A	1	B	1	C	1
D	1	E	1		

STEP 1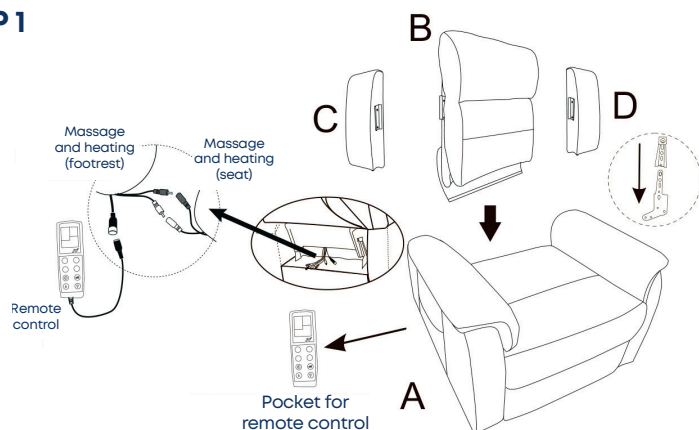

STEP 2

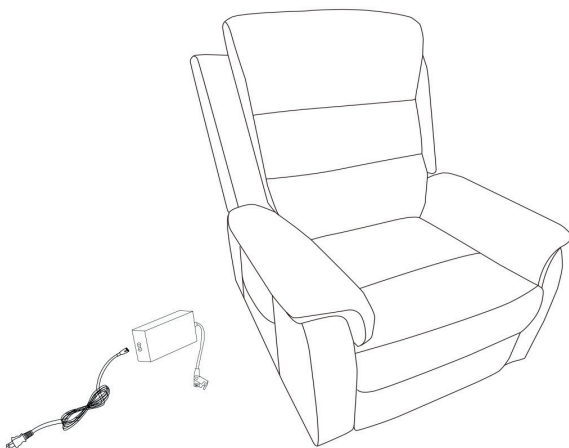ASSEMBLY TIPS:

- Before tightening all screws completely, assemble all parts loosely first. This ensures proper alignment and prevents uneven assembly.
- Once everything fits correctly, tighten all screws securely.
- This assembly may require additional help (2 person set up).

WARNING

- Do not stand in this chair, do not use this chair as a step ladder
- Use this chair only for seating one person at a time
- Do not use this chair unless all bolts, screws and knobs are firmly secured
- Check all bolts, screws and knobs on a regular basis
- This chair should not be used with damaged or missing parts
- Contact and authorized service provider to repair or replace damaged or missing parts
- Failure to follow these warnings could result in serious injury

1. Power button

Press the power button once to turn it on; the red indicator light will illuminate. The four-part massage will activate by default, the blue indicator light will flash, the default timer is 10 minutes, **the massage intensity is M, and the massage mode is 1**. Press it again, the red indicator light will turn off, and the previous functions will also shut down.

2. HEAT: Massage heating

Press once to turn on the heating; the yellow indicator will light up. Press again to turn it off. The heating function operates independently, and the default time is 10 minutes.

3. MODE: Massage mode

There are three massage modes. Select the mode in sequence; the default mode is **M1** when the unit is turned on.

M1 – Mode 1 (blue indicator light)

M2 – Mode 2 (blue indicator light)

M3 – Mode 3 (blue indicator light)

4. BACK: Back massage point switch

The back massage point is activated by default when the unit is turned on, and the blue indicator light flashes. Press again to turn off the back massage points and the indicator light. (Press again to turn it back on).

5. LUMBAR: Lumbar massage point switch

The lumbar massage point is activated by default when the unit is turned on, and the blue indicator light flashes. Press again to turn off the lumbar massage point and the indicator light. (Press again to turn it back on).

6. INTENSITY: Massage intensity

There are three levels of massage intensity:


L - Low / M - Medium / H - High.

The default setting is M when the device is turned on.

You can press the Intensity button to change the setting.

7. THIGH: Thigh massage point switch

The thigh massage point is activated by default when the unit is turned on, and the blue indicator light flashes. Press again to turn off the thigh massage point and the indicator light. (Press again to turn it back on).

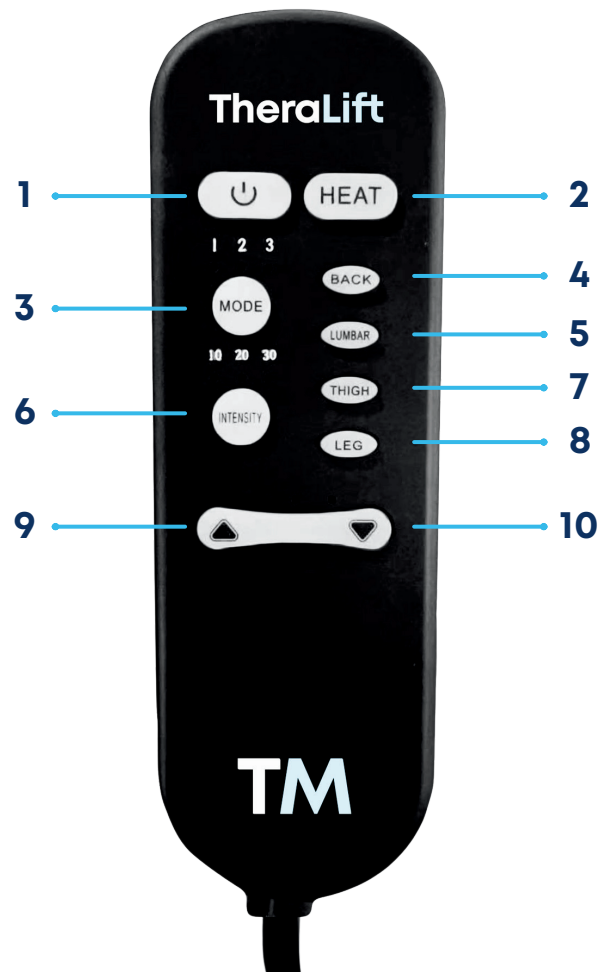8. LEG: Leg massage point switch

The leg massage point is activated by default when the unit is turned on, and the blue indicator light flashes. Press again to turn off the leg massage points and the indicator light. (Press again to turn it back on).

9. UP ▲

Press the push rod once to move upward continuously; press again to stop.

10. DOWN ▼

Press the push rod once to move downward continuously; press again to stop.

TM3000
BROWN

TM3001
BLUEBERRY

TM3002
CREAM

TM3003
MOCCHA

TM3004
NOIR

Product Data

Description

Electric lift chair with 8-point massage and heating.

Up to 3 positions:

- Armchair
- Semi-bed
- Elevation

The armchair can remain in intermediate positions. The position is adjustable according to the user's needs and requirements.

Product parts

- Padded backrest with 3 sections.
- Padded footrests.
- Padded armrest.
- Padded seat.
- Base with rear wheels without brakes.
- Right side pocket.
- Massage and heat control.
- Armchair position control.

Materials

- **Product:** iron frame, motor, wooden frame.
- **Filling:** 30D high elastic foam.
- **Cover:** premium quality fiber, air leather and fabric.

Materials by SKU

- TM3000: Eco-leather upholstery based on PU.
- TM3001: Fabric upholstery based on PU.
- TM3002: Fabric upholstery based on PU.
- TM3003: Fabric upholstery based on PU.
- TM3004: Eco-leather upholstery based on PU.

Maximum supported weight

158 kg

Product dimensions

- **Overall** 84 x 92 x 104 cm
- **Seat** 48 width x 56 depth x 49.5 height cm
- **Armrest height** 60cm
- **Back height when reclined** 79.5 cm
- **Total depth when fully extended** 157cm
- **Footrest height when open** 52 cm

Angle of inclination

Opened 136° / **Closed** 108°

Storage Data

Net weight | Product

40.8 kg

Shipping box dimensions

86 x 76 x 65 cm

Shipping box weight

43.8 kg

Units per shipping box

1

Cleaning instructions

- **Permitted Cleaning Products:** water, mild soap with water, diluted isopropyl alcohol (70%), disinfectant wipes.
- **Cleaning Products Not Recommended:** strong corrosive chemicals, abrasive cleaners, strong organic solvents, high-pressure steam cleaners or direct water rinsing.

Preventive maintenance for eco-leather

- Leather Moisturizer.
- Frequency: Apply every 3 months.

Steps:

- Cleaning: Clean the surface with a soft cloth.
- Application: Apply the leather moisturizer evenly.
- Absorption: Let the moisturizer absorb for 15 minutes.
- Polishing: Buff with a clean, dry cloth.

Storage conditions

- Avoid direct sunlight.
- Dry, ventilated, cool indoor place

Technical Specifications

System

- Electric.
- Only in case of electrical failure or power outage, the chair will automatically return to its original position.

Nominal output voltage

AC100-240V

Input voltage range

DC29V-1.8A

Warranty

The iron frame has a 5-year warranty. The motor, wooden frame, 30D high-elastic foam, premium-quality fiber, air leather, and fabric have a 2-year warranty. *Damage caused by misuse or human error is not covered under warranty.*

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