# TheraClassic

POLTRONA / SILLÓN / CHAIR

**ES** Manual de instrucciones **PT** Manual de instruções **EN** Instruction manual

TM4000

## TheraMart

# TheraClassic

POLTRONA / SILLÓN / CHAIR

## ESPAÑOL

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## Product Features

The TheraClassic recliner has been designed with a focus on convenience and comfort, ideal for both home and hospital environments. This versatile ally is perfect for rooms where long-term treatments such as hemodialysis, hemotherapy and chemotherapy are performed.

With a design that evokes the classic Berger armchair, the TheraClassic has a wide and discreet backrest that integrates harmoniously into any decoration, providing a touch of sobriety and comfort. Its waterproof and easy-to-clean upholstery makes it ideal for the clinical setting, allowing for simple sanitization with just a cloth.

Every detail of the TheraClassic is designed to maximize its functionality: it offers smooth and progressive recline and decline. Its armrests are removable, improving access and transfer for people in wheelchairs or with reduced mobility.

In addition, it is equipped with a rear handle that makes it easy to adjust the position according to the user's needs, and four wheels that allow it to be easily moved between different areas. The rear wheels include safety brakes to ensure the stability of the chair when in use.

### **Warning Symbol**

**Carefully follow the instructions associated with this symbol.**

It is important to pay attention to these indications; Failure to do so could result in personal injury or damage to the chair and the environment.

### **WARNINGS:**

**Safety warnings for using the chair:**

- It is prohibited to stand on the chair or use it as a ladder.
- Individual use: Use this chair only to seat one person at a time.
- Component Integrity: Do not use this chair unless all bolts, screws and nuts are securely secured.
- Regular maintenance: Check all bolts, screws and nuts regularly.
- Using Integer Parts: Do not use this chair if it has damaged or missing parts.
- Repairs: Contact authorized dealer to repair or replace damaged, defective or missing parts.
- Consequences of not following warnings: Ignoring these recommendations could result in serious harm.

## Technical Specifications:

|                  |       |             |        |
|------------------|-------|-------------|--------|
| Product weight   | 37kg  | Seat width  | 50.5cm |
| Gross weight     | 42kg  | seat height | 51cm   |
| Supported weight | 130kg | Seat depth  | 52cm   |

|                                                    |       |                          |                |
|----------------------------------------------------|-------|--------------------------|----------------|
| Overall width                                      | 76cm  | Height from floor to arm | 70cm           |
| Overall Height                                     | 110cm | Carton size.             | 82 x 77 x 78cm |
| Full length in sleeping position or fully reclined | 155cm | Material:                | PVC            |

|                                           |     |                                           |      |                                             |      |
|-------------------------------------------|-----|-------------------------------------------|------|---------------------------------------------|------|
| Degrees of inclination in normal position | 90° | Degrees of inclination in middle position | 120° | Degrees of inclination in reclined position | 140° |
|-------------------------------------------|-----|-------------------------------------------|------|---------------------------------------------|------|

## Assembly instructions

|                                      |                                                                                                        |                                                                                       |
|--------------------------------------|--------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|
| <b>Handrail installation:</b>        | Raise the back of the chair and move it to the side to allow room to install the handrail or handrail. |    |
|                                      | Place the grab rail on the back of the backrest until it fits snugly.                                  |  |
| <b>Assembly of the folding arms:</b> | Remove the folding arms that are stored under the chair seat.                                          |  |

|                            |                                                                                                                                                                                                                                                                                   |                                                                                      |
|----------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------|
|                            | <p>Insert your arms into the base on the side of the seat, applying gentle pressure. Make sure the steel rods at the bottom of the arms fit correctly into the holes on the sides of each side of the seat. These arms are removable and should be easy to remove and attach.</p> |   |
| <b>Backup Installation</b> | <p>Carefully hold the base of the backrest without placing your fingers on the rails or sockets until they are aligned with the metal piece located just behind the seat.</p>                                                                                                     |  |
|                            | <div data-bbox="630 1285 743 1388" data-label="Image"> </div> <p><b>WARNING:</b> Avoid accidents: Do not place your fingers in the metal fittings of the backrest when installing it on the seat.</p>                                                                             |                                                                                      |

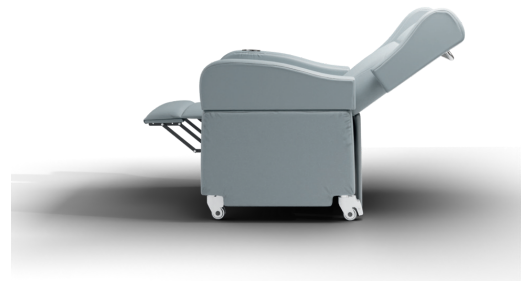
|  |                                                                                                                                                                                                                                                                                                        |                                                                                     |
|--|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|
|  | <p>Once aligned, allow the backrest to lower under its own weight and, applying pressure to the top, adjust until you hear a click. This will indicate that the backrest and seat of the TheraClassic are correctly connected. Now, the TheraClassic is ready to use and provide the best comfort.</p> |  |
|--|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|

Extra: Use the magic closure on the fold of fabric protruding from the base of the backrest to adjust it to the base of the seat

## Instructions for Use

The TheraClassic is a manual reclining chair that is easily adjusted by body movement, offering up to 3 different positions for your comfort.

1. **To sit and recline slightly:**
  - a. Sit on the couch.
  - b. Use your own weight, pressing on the part of the chair behind your calves and against the backrest, to move it to an intermediate position that will allow you to raise your legs and recline slightly.
2. **To fully recline the chair:**
  - a. Use your back to push the backrest into a sleeping position, taking advantage of the chair's intuitive design that responds to your body's natural movement.
3. **To return the chair to its normal position:**
  - a. Apply enough pressure on the backrest and legs so that the chair returns to the initial position under its own power.

- b. If you need assistance, ask for help using the handrail or handle behind the backrest. By applying pressure on it, the mechanism will be activated that allows the chair to return to its original position.

## Cleaning and Care Instructions

1. Wipe regularly with a soft, dry cloth to remove dust and dirt.
2. For light stains, use a cloth slightly dampened with warm water and neutral soap.
3. Never use abrasive cleaning products, solvents or harsh chemicals.
4. Avoid direct exposure to sunlight and excessive heat to protect the material.

## Warranty

We offer 2 years warranty for main parts such as mechanism and wooden frame, and 1year warranty for all upholstery fabrics.

## Precautions

### **Do not use organic solvents**

Under no circumstances should any type of organic solvent (such as petroleum, alcohol, etc.) be used to clean the chair. The use of these products could damage the materials and compromise the integrity of the chair.

### **Avoid sharp or sharp objects**

Nor should any pointed or sharp object be used to clean the chair. This could cause damage to the upholstery or upholstery surfaces.

### **Supervision of minors and vulnerable people**

This chair is not intended for use by children, or people with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless supervised by a responsible person. Adult supervision is essential to ensure safe use of the chair.

**Do not allow children to play with the chair.**

For their own safety, children should be supervised by adults to ensure that they do not play with the chair. Improper use can cause accidents or damage the chair.

For indoor use only.

Please refer to the user manual before use.

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Fabricado na China

---

TheraMart is a brand of Grupo Thera INC.

Oceania Business Plaza, 43rd Floor, Punta Colón St., Panama City, Panama.

Contact information for the rest of the world: [info@theramart.com](mailto:info@theramart.com)

Designed in LATAM

Made in China

TM4000

# TheraMart