
TheraMart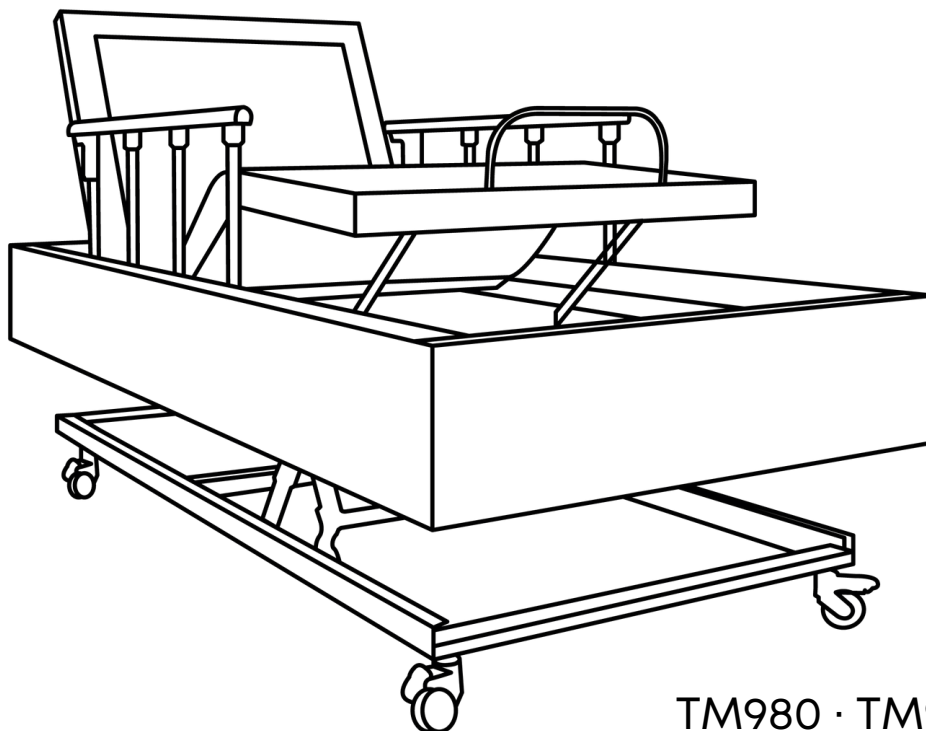

TM980 · TM981

EN

INSTRUCTION MANUAL

TheraFlex Bed

PT

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Cama TheraFlex

ES

MANUAL DE INSTRUCCIONES

Cama TheraFlex

V5 / 27 MAR 2026

[EN]

This instruction manual will present precautions and the safe use and operation of this bed. Before using this bed, please read this instruction manual carefully to fully understand the safe and correct operating procedures.

The correct operating procedures should not only be explained to the person using this bed but also communicated to the caregiving staff.

After reading, please keep this instruction manual properly for future reference.

If you have any questions, please contact TheraMart directly.

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SUMMARY | SUMÁRIO | CONTENIDO

[EN]

1. SAFETY INFORMATION	4
SAFETY INFORMATION	4
TIP-OVER RISK WARNING	5
INTENDED USE	5
USER OPERATION AND SUPERVISION	5
PACEMAKER COMPATIBILITY	5
MAXIMUM WEIGHT CAPACITY	5
OPERATING NOISE	6
FCC COMPLIANCE	6
2. INTRODUCTION	6
FEATURES	6
CAPABILITIES	6
3. INSTALLATION GUIDE	6
UNPACKING	7
POWER CORD ASSEMBLY	7
INSTALLING THE CASTERS	7
BED POSITIONING	8
PLACEMENT	8
HANDRAILS	8
INSTALL THE MATTRESS RETAINER BAR	8
REMOTE CONTROL BATTERY INSTALLATION	8
4. REMOTE	9
5. TROUBLESHOOTING	9

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[ENGLISH]

1. SAFETY INFORMATION

PLEASE READ ALL SAFETY INFORMATION AND INSTRUCTIONS CAREFULLY BEFORE ASSEMBLY AND USE. VIOLATING ANY SAFETY INFORMATION WILL VOID THE WARRANTY AND MAY RESULT IN INJURY OR DAMAGE.

SAFETY INFORMATION

Use the Theraflex bed only for its intended purpose, as described in this manual:

- Avoid placing fingers in joints or pinch points.
- It is recommended that two people perform the moving and assembly.
- Connect the adjustable bed to a proper outlet or a surge protector (not included).
- If the plug does not fit the outlet, have a qualified electrician install a suitable outlet.
- Do not make unauthorized modifications or use improper power sources. This may void the warranty.
- Keep power cords away from heated surfaces.
- To safely disconnect, place the bed in a flat position, turn off all motors, and unplug it.
- Disconnect the product from the outlet before cleaning or servicing.
- Stop using the bed and contact Customer Service if it is not working properly, has been exposed to water, or if the power cord is damaged.
- Do not use the product near water or liquid sources.
- Never operate the adjustable bed with objects or obstructions between moving parts and the metal frame.

- Replacement of parts may be performed by the consumer, but any alteration of internal components will void the warranty.
- Do not use accessories or parts not recommended by the manufacturer.
- Weight limit: 375kg, including mattress and users. Exceeding this limit may cause damage or accidents and void the warranty.
- Do not use the product when people or animals are under the adjustable bed.
- The mattress may shift on the adjustable base – use with caution.
- If adjustable legs are used, make sure all are properly fixed and resting on the floor. Do not use legs taller than 4 segments, as this may compromise safety.

TIP-OVER RISK WARNING

Avoid placing excessive weight on the edges of the bed, as this may cause it to tip over.

INTENDED USE

The Theraflex Bed was designed to provide comfort and support to the user.

USER OPERATION AND SUPERVISION

Use of the Theraflex bed by children, pets, or convalescent persons must be supervised.

Dispose of all packaging materials immediately, as they may pose a choking hazard to children and pets.

Do not allow children or small animals to play on or under the adjustable bed.

PACEMAKER COMPATIBILITY

Since this product generates vibratory motion, some pacemakers may misinterpret the stimulation. If in doubt, consult your physician.

MAXIMUM WEIGHT CAPACITY

The Theraflex bed supports up to 375kg, including the mattress and users, evenly distributed across the surface.

This product is not designed to support the entire weight solely at the head or foot sections.

Do not enter or exit the bed when the head or foot sections are elevated. Always return the bed to a flat position before use.

If the weight limit is exceeded on the head or foot mechanisms, the control box will automatically shut down all functions.

Exceeding the weight limit may cause damage to the adjustable bed, pose a risk of accidents, and void the warranty.

OPERATING NOISE

The massage motor emits an audible sound during operation, which may vary depending on intensity and floor type.

Minor metallic noises during bed articulation are considered normal.

FCC COMPLIANCE

This device complies with Part 15 of the FCC Rules, subject to the following conditions:

- 1. It may not cause harmful interference.*
- 2. It must accept any interference received, including interference that may cause undesired operation.*
Radio frequency: 2.4 GHz.

2. INTRODUCTION

FEATURES

- Wireless Remote
- Head up/down
- Foot up/down
- Flat Position Preset
- Zero Gravity Preset
- Parallel lifting
- Anti-Snore Preset
- Mattress Retainer Bar

CAPABILITIES

- Weight limit: 375 kg. including the mattress and users.

3. INSTALLATION GUIDE

1) UNPACKING

Place the box on a flat surface.


COMPONENTS

- (1) Remote
- (2) AAA Batteries
- (4) Wheels
- (1) Power Cord
- (1) AC/DC Power Adapter


- (1) Power Cord
- (1) Mattress Retainer Bar
- (2) Handrail
- (1) Allen Wrench
- (8) Screws
- (1) Wrench

Mattress Retainer Bar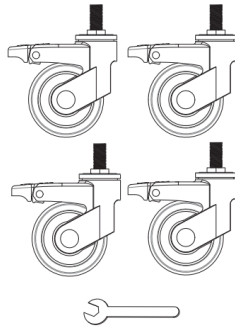

Wheels

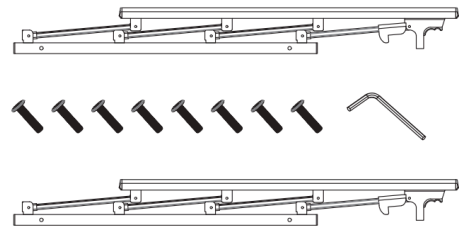
Handrail

2) POWER CORD ASSEMBLY


Locate the control box built into the underside of the adjustable base. Connect the control box power cord, the AC/DC Power Adapter, and the Power Cord.

Store the Emergency Battery Strip in a safe location in case of a power outage; two 9-Volt batteries are required for operation and are not included.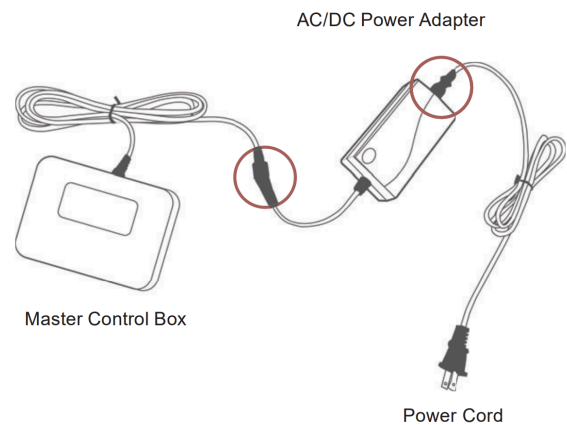


3) INSTALLING THE CASTERS

- Loosen the pre-assembled nuts on the casters.
- Attach the casters to the 4 designated points using the supplied screws and nuts.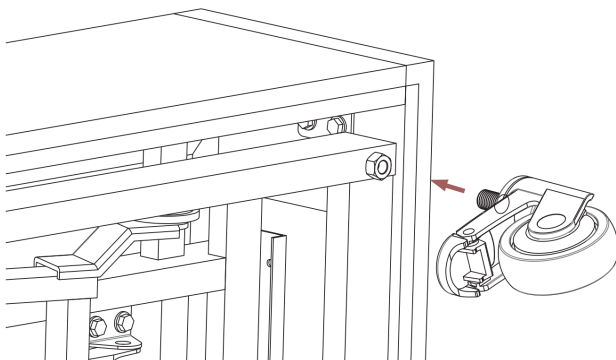

4) BED POSITIONING

- Place the bed in an upright and stable position.
- Connect the power cord to a proper electrical outlet.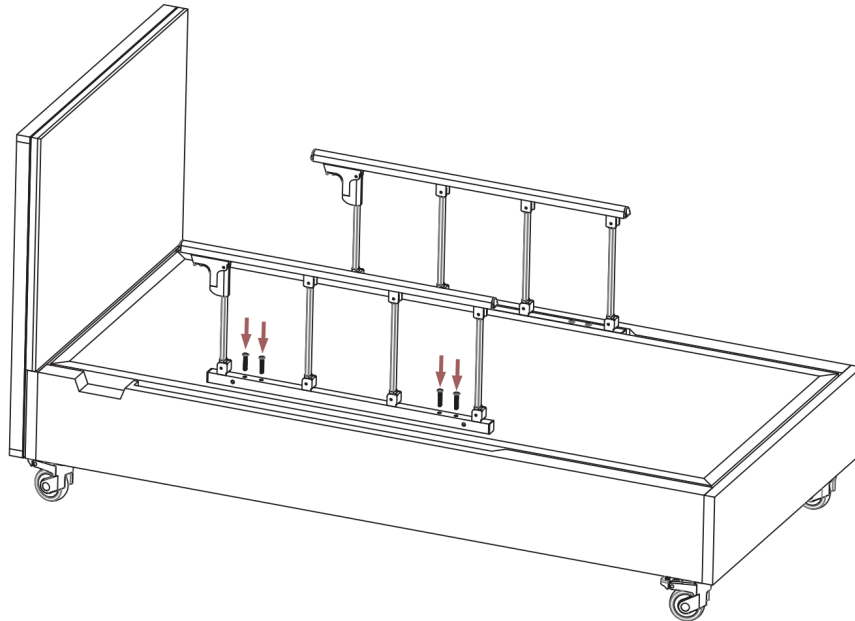


5) PLACEMENT

Return the adjustable base to an upright and secure location, then plug in the power cord to an outlet.

6) HANDRAILS

Attache the handrails to both sides of bed frame by screws.

7) INSTALL THE MATTRESS RETAINER BAR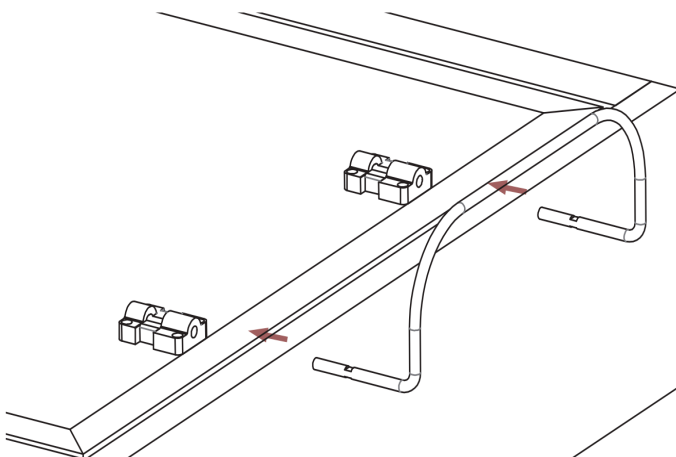


8) REMOTE CONTROL BATTERY INSTALLATION

The remote control is pre-programmed to operate the adjustable bed.

If you have more than one bed, each remote will work independently.

4. REMOTE

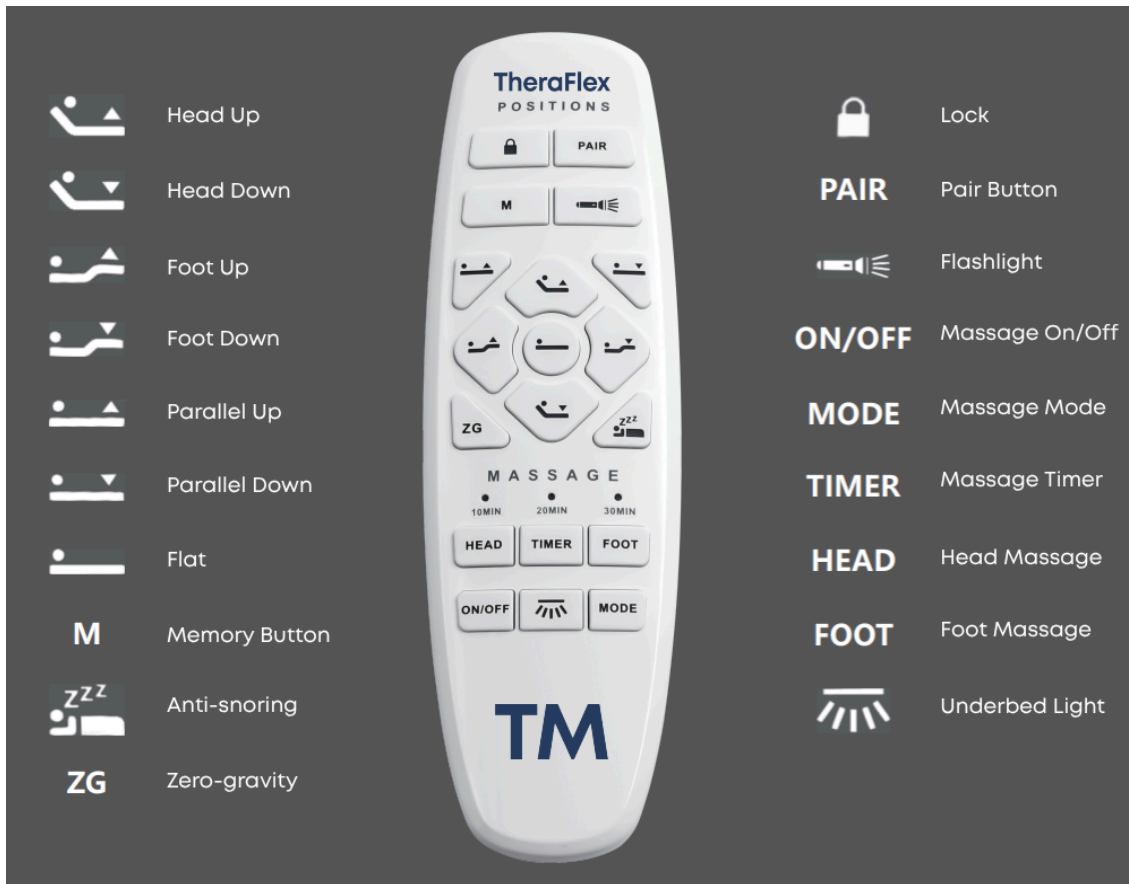5. TROUBLESHOOTING

IF ONE OR MORE FUNCTIONS ON THE ADJUSTABLE BASE HAVE STOPPED OPERATING

- Check under the adjustable base to verify that the wired connections are secure and that there are no cords or bedding obstructing the movement of the adjustable base.
- Check if the power cord is assembled properly and all the connections are secure.
- Check if the green LED is on the AC/DC power adapter is illuminated.

IF THE LIGHTS ON YOUR REMOTE CONTROL NO LONGER ILLUMINATE AND WILL NOT OPERATE YOUR ADJUSTABLE BASE

- Make sure the batteries are installed correctly.
- Install two new AAA batteries in the remote control.

IF THE REMOTE CONTROL BUTTONS ILLUMINATE BUT ADJUSTABLE BASE WILL NOT OPERATE

- Unplug the adjustable base for 5 minutes to reset the electronic components.
- Plug the adjustable base into a different electrical outlet, or test the current outlet with another working appliance (a grounded, electrical surge protector is recommended).
- If the steps above do not fix the issue, you can reset your remote frequency by following the REMOTE FREQUENCY RE-PAIRING in the troubleshooting section.

IF YOUR PRESET POSITION IS NOT WORKING PROPERLY

- Reset your remote preset positions by holding down the zero gravity and flat position button for 7 seconds.

IF THE HEAD OR FOOT MECHANISMS WILL ELEVATE BUT WILL NOT RETURN TO THE FLAT POSITION

- Check the undercarriage for obstructions. Remove any obstructions away from the head and foot mechanisms and return the adjustable base to its flat position.

REMOTE FREQUENCY RE-PAIRING

The remote is pre-programmed to the adjustable base, however if the remote is unresponsive, follow these steps:

- Check the batteries; replace if needed.
- If the remote illuminates: Unplug the power cord from the outlet.
- Wait 5 seconds and plug it back into the outlet.
- Within 10 seconds of plugging the power cord into the outlet: Press the "PAIR" button at the same time for 10 seconds, the adjustable base will be re-paired with the remote.

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